
Brand: AUDINELL
Ref: 4 PDO-59_A04
Date: 26/06/2024

Software used: IN-DESIGN CC 2024
Print type: offset or digital
Print media: white paper 70 gsm offset

Colors used:

CMYK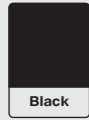

Black


VARNISH


None

Texts specifications:


Open format


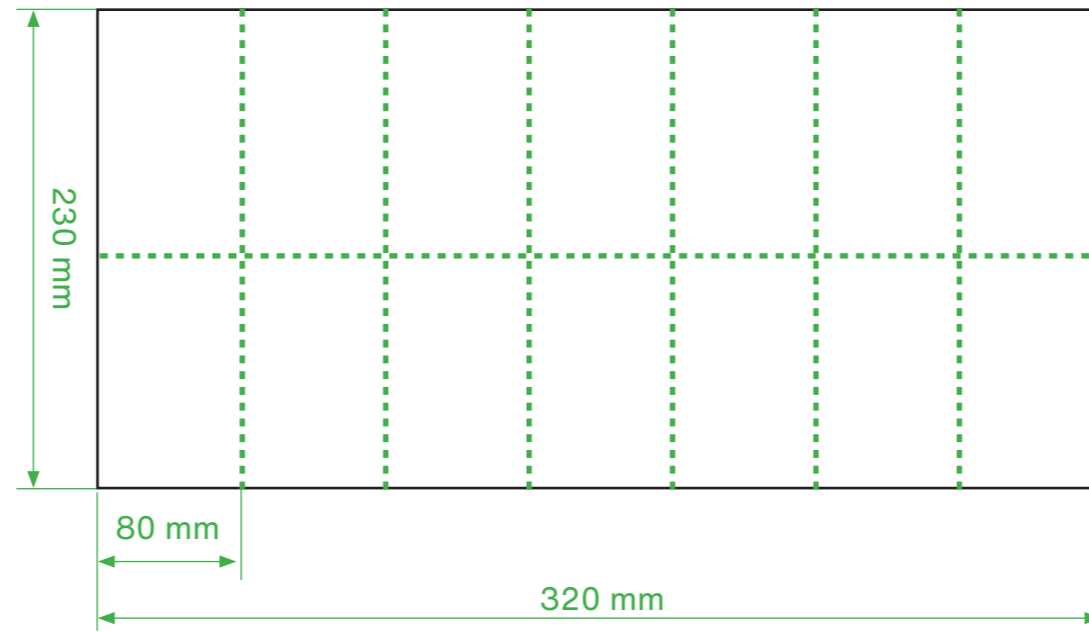


Folding -----

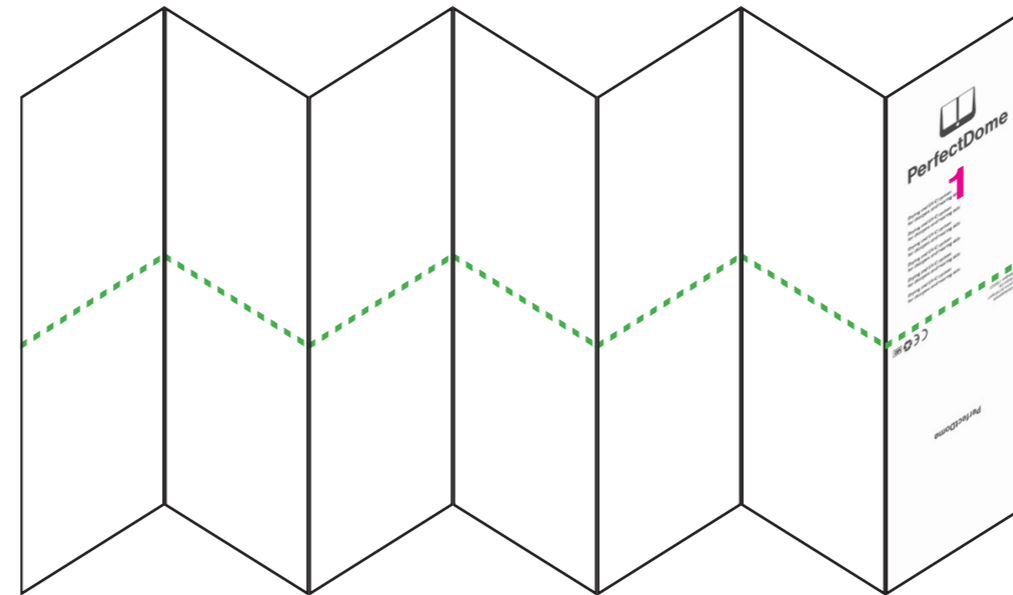
Closed format

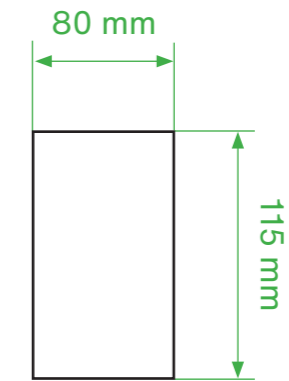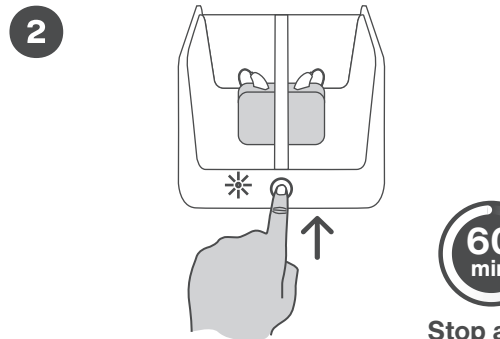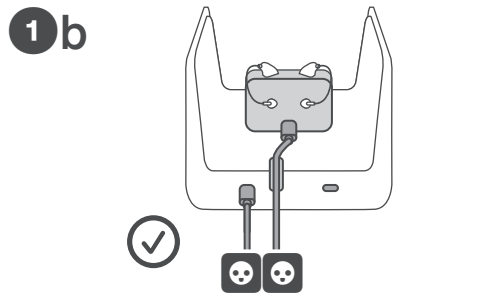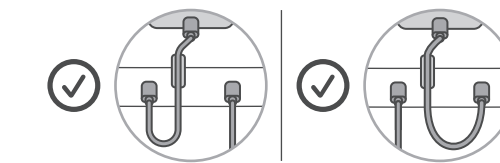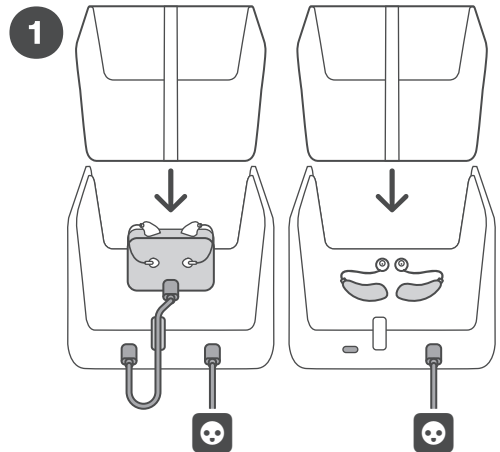Stop auto.
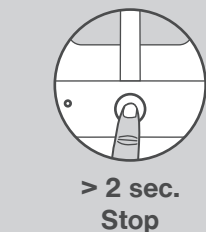
> 2 sec.
Stop

Stop

EN

1 Connect PerfectDome. The white light will flash 3 times to indicate that the power is on. Connect your hearing aid charger to PerfectDome. If the charger is not working, connect it directly to its original power supply (Fig. 1b). PerfectDome is suitable for all types of hearing aids and related chargers.

2 To start the cycle, touch the touch-sensitive button (cover closed). The blue light will flash slowly. PerfectDome switches off automatically after 60 minutes or after pressing the touch-sensitive button for 2 seconds. The blue light switches off. Also make sure you do not open the cover while it is operating, as PerfectDome stops working if the cover is opened. It is recommended cleaning the hearing aids before the dehumidification.

Function details

UV-C

PerfectDome is fitted with UV-C LEDs positioned to reach all the surfaces to be treated. The UV-C LEDs operate throughout the maintenance cycle.

Dehumidification

PerfectDome features a patented pulsed hot air dehumidification system. The maintenance temperature is controlled by an electronic system enabling it to adapt to the conditions of use of the hearing aids.

To maintain the optimal quality of your hearing aids, the daily use of PerfectDome is highly recommended.

Intended use:

PerfectDome is a Class I medical device designed for the maintenance of hearing aids on their own or in their respective chargers. PerfectDome has been designed to:

Dry hearing aids to prevent them from malfunctioning due to moisture. Treat the surface of hearing aids to remove any potential micro-organisms, thereby reducing the likelihood of suffering ear canal problems associated with wearing hearing aids.

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Intended users/groups of users:

Hearing-impaired people who use hearing aids and their carers. Hearing-aid specialists caring for the user. Adults and children aged 16 years and older.

Safety instructions

Keep out of the reach of children and people with cognitive disorders. Do not immerse PerfectDome and its accessories in water or any other liquid. Replace your mains adapter with an original model. Place PerfectDome on a flat, dry and stable surface. Failure to comply with these instructions will invalidate the guarantee. The manufacturer shall not be held liable for any damage caused by improper use or mishandling.

If you experience problems, contact a hearing aid specialist. All serious incidents related to the device should be the subject of a notification to the manufacturer and the competent authorities in the Member State in which the user is located.

Fault

The red light comes on. Disconnect and then reconnect PerfectDome. If the problem persists, contact your hearing-aid specialist.

Product care advice

Clean using a soft cloth. Do not clean the PerfectDome using abrasive products. Do not spray liquids into the device. Keep the PerfectDome closed when not in use to avoid impurities from being deposited.

Regulation applicable to Medical devices

This product meets the requirements of Regulation (EU) 2017/745 covering medical devices.

Guarantee

24 months - (including cables and transformer). In the event of a malfunction, contact your hearing-aid specialist. Proof of purchase will be requested for all work by the customer service during the guarantee period. This guarantee does not apply to defects caused by an accident or incident or similar damage, by introducing a liquid, negligence, unsuitable use, dismantling of any kind, lack of care or any other circumstance caused by the user. Furthermore, this guarantee will not apply to damage caused by lightning or any other electric voltage variation. As a precaution, please disconnect the PerfectDome in the event of a thunderstorm.

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Identification data

Meaning of abbreviations:

CE This product is compliant with Regulation (EU) 2017/745 on medical devices.	
UK Conformity Assessed	
Manufacturer	Date of manufacture
UDI Unique device identifier	MD Medical device
SN Serial number	Read the leaflet carefully
REF Catalogue reference	5V-2A Direct current
LOT Batch number	Indoor use only
The product is made in ABS with a recycling code of 7	Sorting instructions
DISPOSAL - Dispose of old appliances ecologically. Old devices contain recyclable materials which must be recovered. Therefore, make sure old devices are disposed of using the appropriate collection systems.	

Technical data

Functions

External (twxwd): 157 x 122 x 140 mm	Drying temperature: +35 °C (+95 °F) max.
Weight (g): 419 g	Cycle duration: 60 min.
LED UV-C: 275 nm	Ambient operating temperature: From +10 °C (+50 °F) to +35 °C (+95 °F)
Max. power: 10 W	Power Supply: Input: 100-240 Vac 50-60 Hz / Output: 5 Vdc
Standby power: 0,05 W	Connectors: USB-A - USB-C cable
Storage conditions from -20 °C (-4 °F) to +65 °C (+140 °F)	

I protect the environment, I disconnect my PerfectDome once my hearing aids are clean!

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