



DIRECTIONS FOR USING DRI-EZE HEARING AID DEHUMIDIFIER:

Prior to first use insert the protective foam pad into the bottom of the Dri-Eze. Write the replacement date (6 months from first use) in the space provided (see bottom of label on jar under written instructions) to indicate when new Dri-Eze Dehumidifier is needed.

1. Before retiring for the night, clean your hearing aids (and ear molds if you have them) with AudioWipes™ cleaning wipes
2. Open battery doors
3. Place the hearing aids into the Dri-Eze
4. Close the lid
5. Remove hearing instruments the next day and be sure to close the lid
6. Repeat this process every night until the end of 6-months when it is time to replace with a new Dri-Eze

IMPORTANT: KEEP LID CLOSED AT ALL TIMES OTHER THAN WHEN INSERTING OR REMOVING HEARING INSTRUMENTS TO ENSURE 6-MONTH EFFECTIVENESS

NOTE: There is no need for recharging or reactivating the desiccant lining. At the end of 6-months, simply replace with a new Dri-Eze Dehumidifier