

xrai
ar one

User guide

Getting started with XRAI Glass® on the AR One



Get started



Need help or support?
WhatsApp: +1 (862) 352-7009

v2.5

Getting started

Step one

Download the app for free

Download the XRAI Glass® app for free on either your Android or iOS device.¹ Once downloaded, sign in using your social login and complete the short onboarding.

Download now

¹ XRAI Glass requires Android 11+ or iOS 13.7+




You'll need to turn your Bluetooth on when first connecting the glasses to your mobile device.



You might need to charge your glasses before turning them on for the first time.



Read more...
Signing in to the XRAI® Glass app

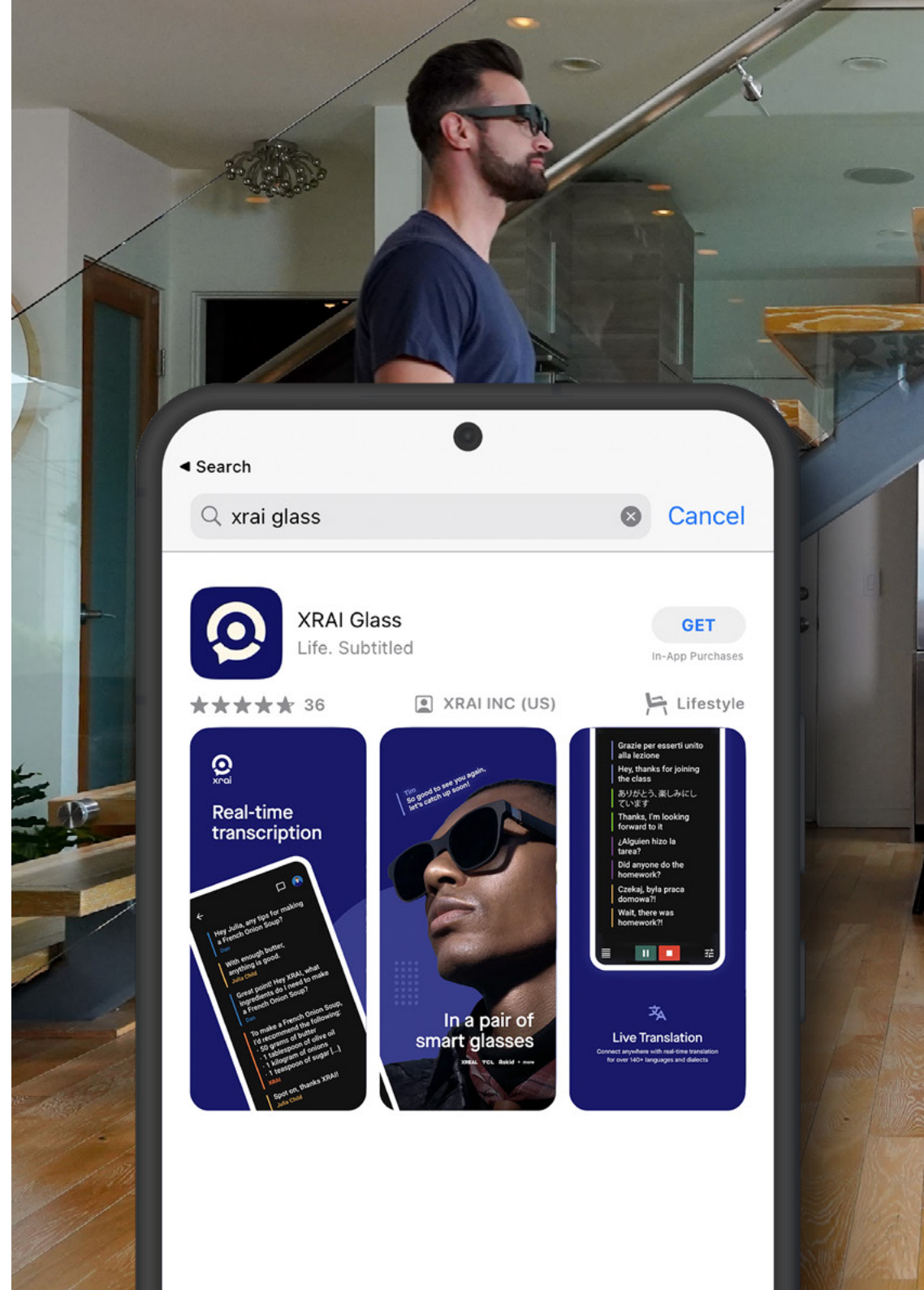

Getting started


Step two

Update your glasses

To ensure you have the best experience right from day one, we recommend updating your glasses as soon as they arrive. This ensures you have the latest features, improved performance, and enhanced security.

How to update

1. Go to <https://link.xrai.glass/ar-one> on a device you can connect your glasses to - in most cases, this will be a desktop computer with a USB to USB-C cable.
2. Connect your glasses with the online updater tool.
3. Scroll down the options on your right until you reach the **Update Firmware section**. There are also some other options to manually adjust the brightness and time zone of your glasses.
4. Once connected, you can click update now on the firmware to ensure you have the latest version of the software running your glasses.

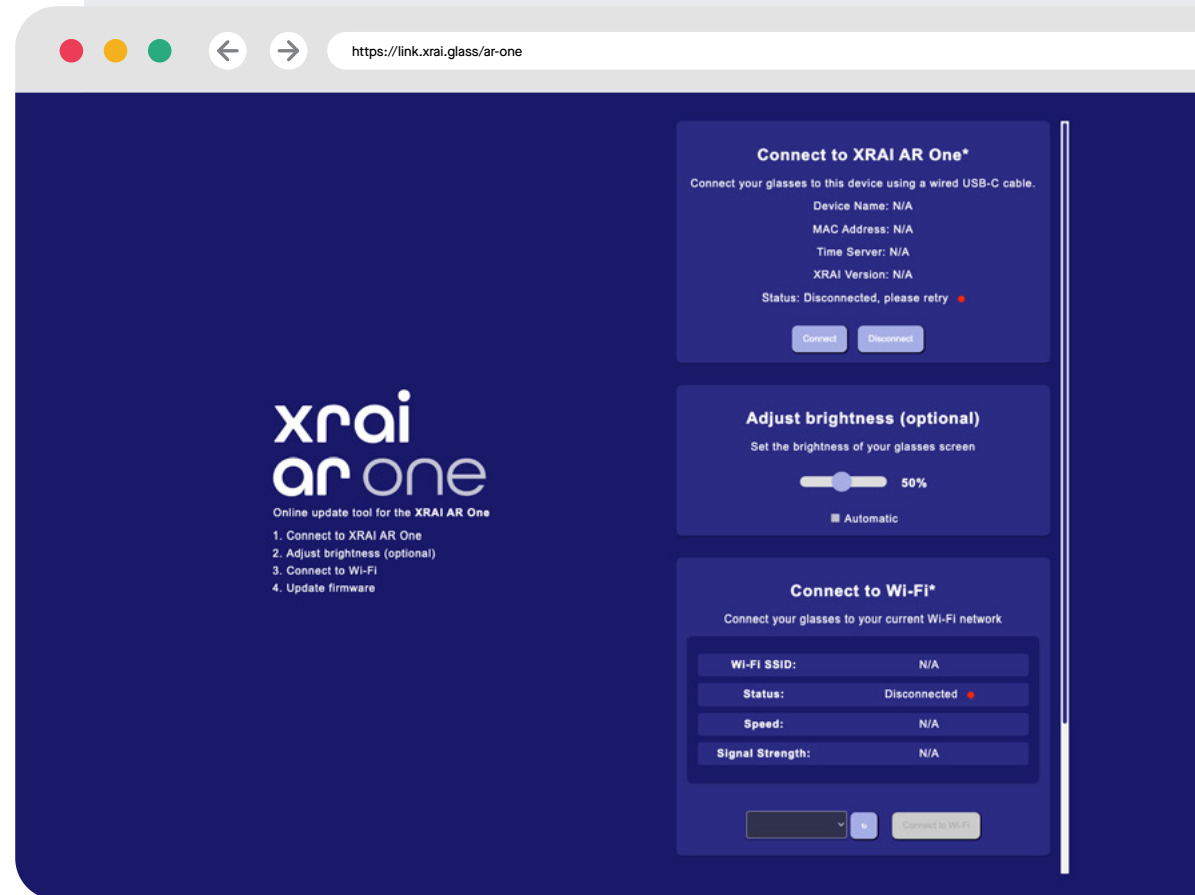

Read more...

[How to update your XRAI AR One glasses](#)

Getting started

Step three

Get your pairing code

After updating, restart your glasses and wait for the XRAI Glass® app to open automatically. Once open, you should see a unique 6 digit code in front of your eyes. If not, you can reach this screen by pressing the function button 3 times in a row.



Press 3 times



Hold for 3-5 seconds



Step four

Pair your devices

Head back to your mobile device and click the  **pairing button** in the top right corner of your screen.

Next, select the Leion Hey glasses from the **Nearby devices** menu. Remember, you'll need to have your Bluetooth turned on to discover nearby devices.


Finally, enter your WiFi details and pairing code to connect the glasses to your device.¹ You can find your pairing code in your glasses from the previous step. Once connected, you'll be able to use your mobile device to control the settings of the XRAI Glass® app within your glasses.

¹ After device pairing is completed, you'll be able to turn off your Bluetooth.



Using your glasses on the go?

Simply disconnect from the WiFi, make sure you're connected to your 4G/5G network, then set up a hotspot on your iOS or Android device. You can then use your hotspot details to pair your devices.

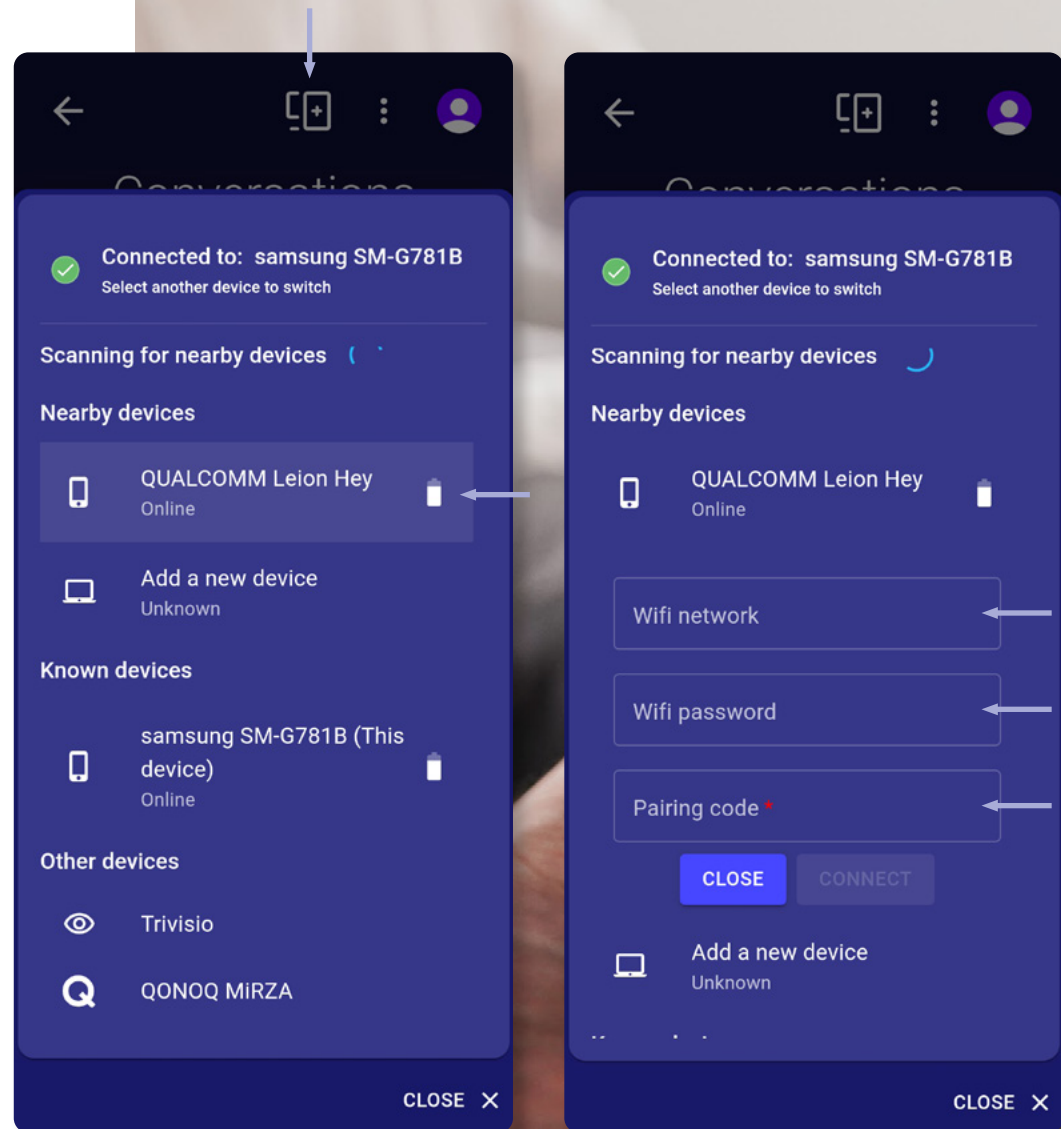



Overview

Getting to know the XRAI AR One smart glasses



Overview

Getting to know the AR One

1. Power button
2. Function button
3. LED Indicator
4. High-performance microphone
5. Ambient light sensor
6. Internal battery
7. Optical waveguide display
8. Adjustable nose pads
9. USB-C port

Power button (1)

Single tap = sleep / wake device

Function button (2)



Single tap = start / pause* subtitles

Double tap = stop subtitles

Triple tap = pairing screen

* Pausing subtitles will not stop the app.

Make sure to stop your subtitles to put your glasses into sleep mode.

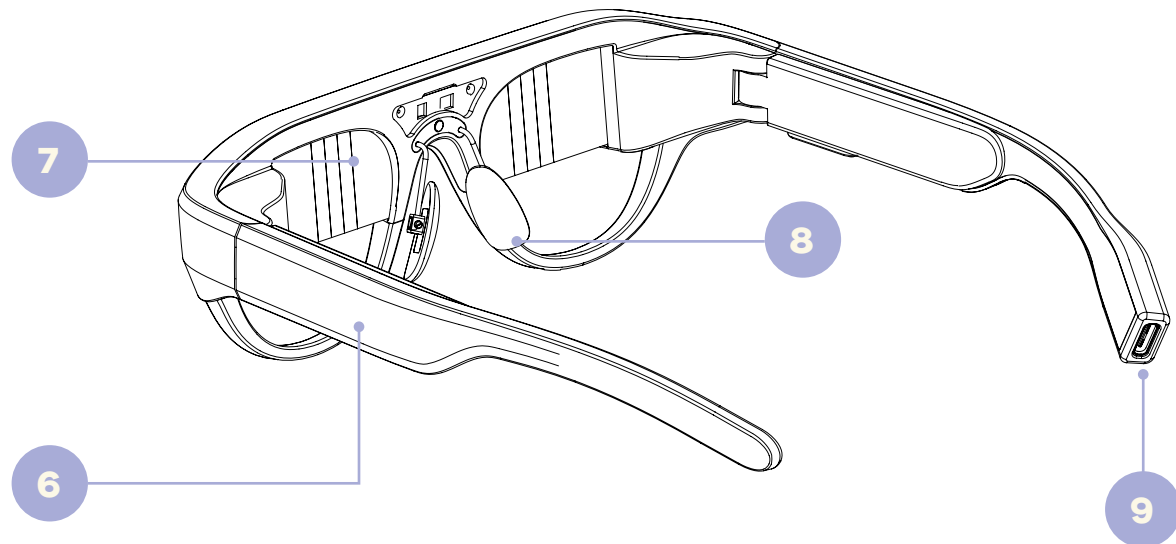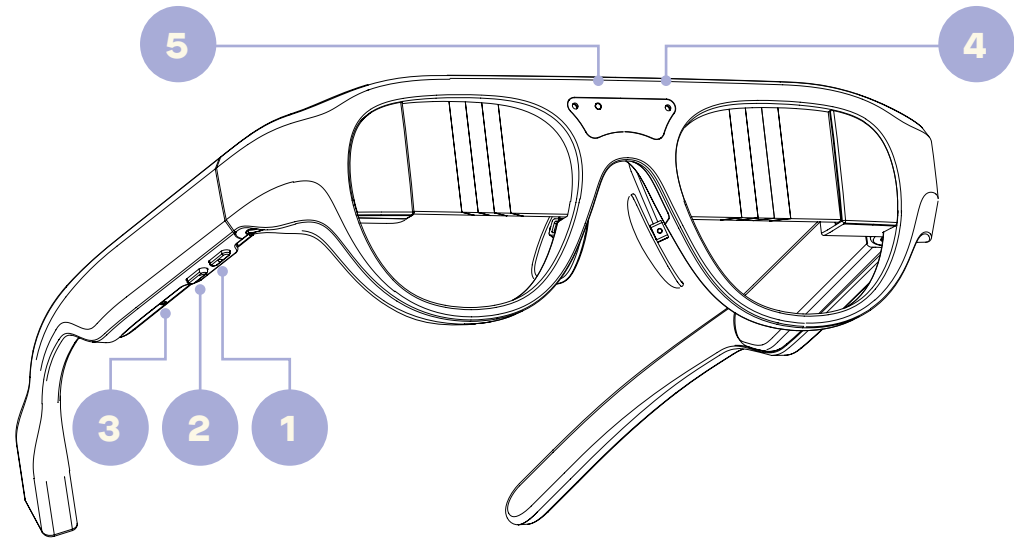

Prescriptions

How to fit custom lenses



Your glasses come complete with a pair of blank clip-on lenses that can be customized to your prescription. Simply send the clip-on lenses to your optician or optical specialist to have them tailored to you.

Installation

1. Remove the cover plate located on the inside of your glasses, just above the nose pads.
2. Align your prescription lens insert with the opening slot, with the clip facing away from you. Then simply push in - you should hear a click once attached. To remove, grab either side of the lenses, then pull backwards until the lenses are detached.

Not sure where to go?

We've partnered with [Lensology](#), who provide custom lens inserts for augmented reality glasses (including ours). Simply visit their website, select your glasses, input your prescription, and have them shipped globally straight to your door.

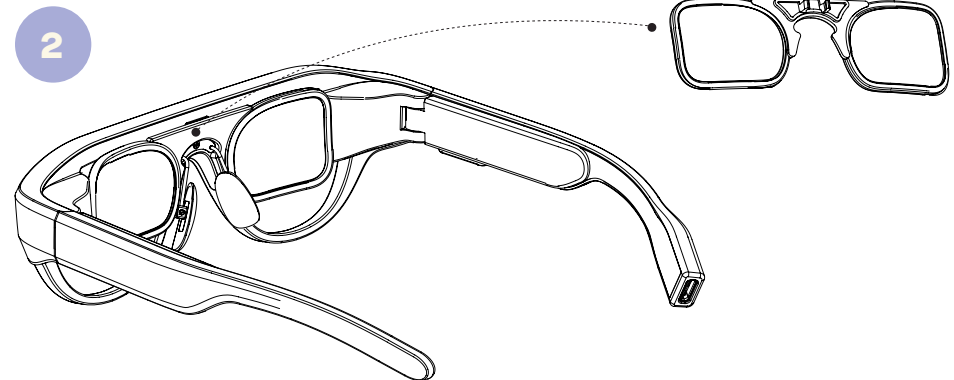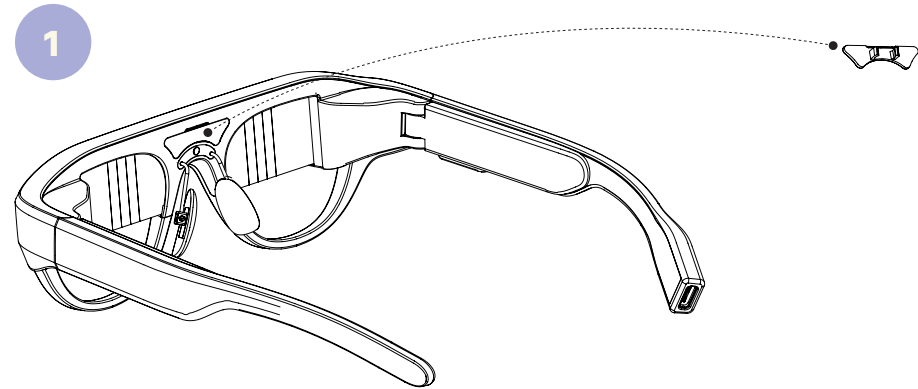



App features

See what you can do with XRAI Glass®



Your account

Sign-in to the XRAI Glass® app

Frustrated with remembering your various logins? We've made it so you can access the XRAI® Glass app using your existing social accounts.

1. Once you open the app for the first time, you'll be prompted to sign in with the **Sign In** button.
2. Next, you'll be prompted to sign in using one of your existing social accounts. Currently, we accept logins from **Apple**, **Google**, **Microsoft**, **LinkedIn**, and **Microsoft 365** accounts. Once signed in, you'll go through a quick-start tutorial to get you up and running as quick as possible.




Microsoft 365 accounts are for enterprise users. Speak with your IT team for more details.



This is what augmented reality is made for.

∞ Meta

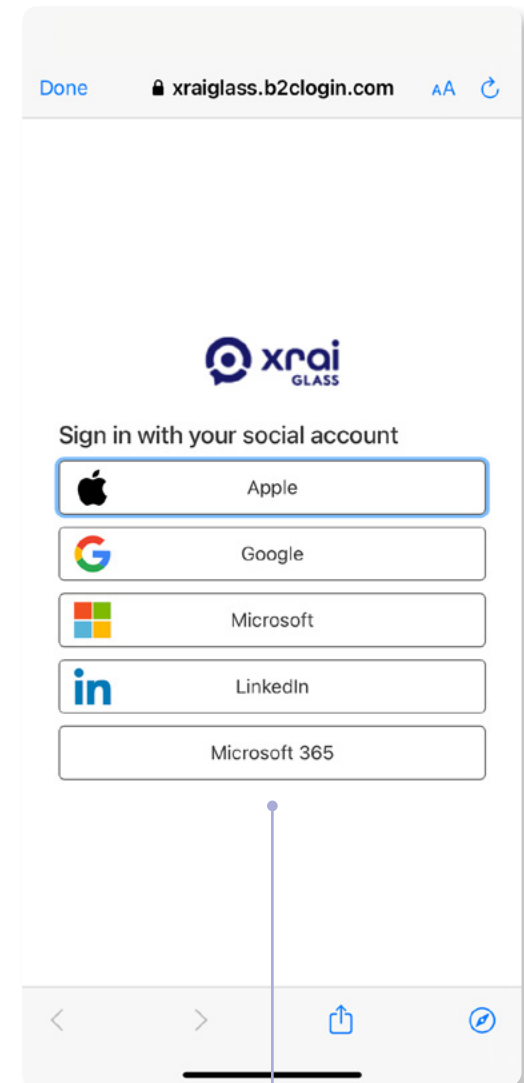

Getting started

Shortcuts

Essential quickstart tips

A few shortcuts to help you find your way in the XRAI® Glass app for the first time.

1. Start a conversation
2. Start a conversation on smart glasses¹
3. Exit app
4. Device pairing
5. Profile / Usage / Settings
6. Back to home
7. Language menu
8. Start / Stop transcribing
9. Quick settings menu

¹ Android devices only.

“

More than just a gadget, the technology helps those suffering from a range of health issues.

WIRED



Transcription

Lightning-fast subtitles


Using the latest cloud-enhanced transcription engines, our app transforms spoken words into written text effortlessly, providing you with a convenient way to transcribe your live audio recordings on the go.

Try it out

1. Start a new conversation and click on the quick settings menu in the bottom right of your screen.
2. If you're having a conversation with just the one speaker, try enabling **Noise cancellation** to help improve the accuracy of your subtitles.
3. By default, your app will use your mobile device as your audio source. If you've got a Bluetooth microphone, try connecting it to your device and changing the **Selected audio source** in the app.
4. If you're using a compatible pair of augmented reality glasses, try enabling the **Minimal interface** to change the interface to focus more on your conversation.



Read more...
[What is cloud-enhanced transcription?](#)

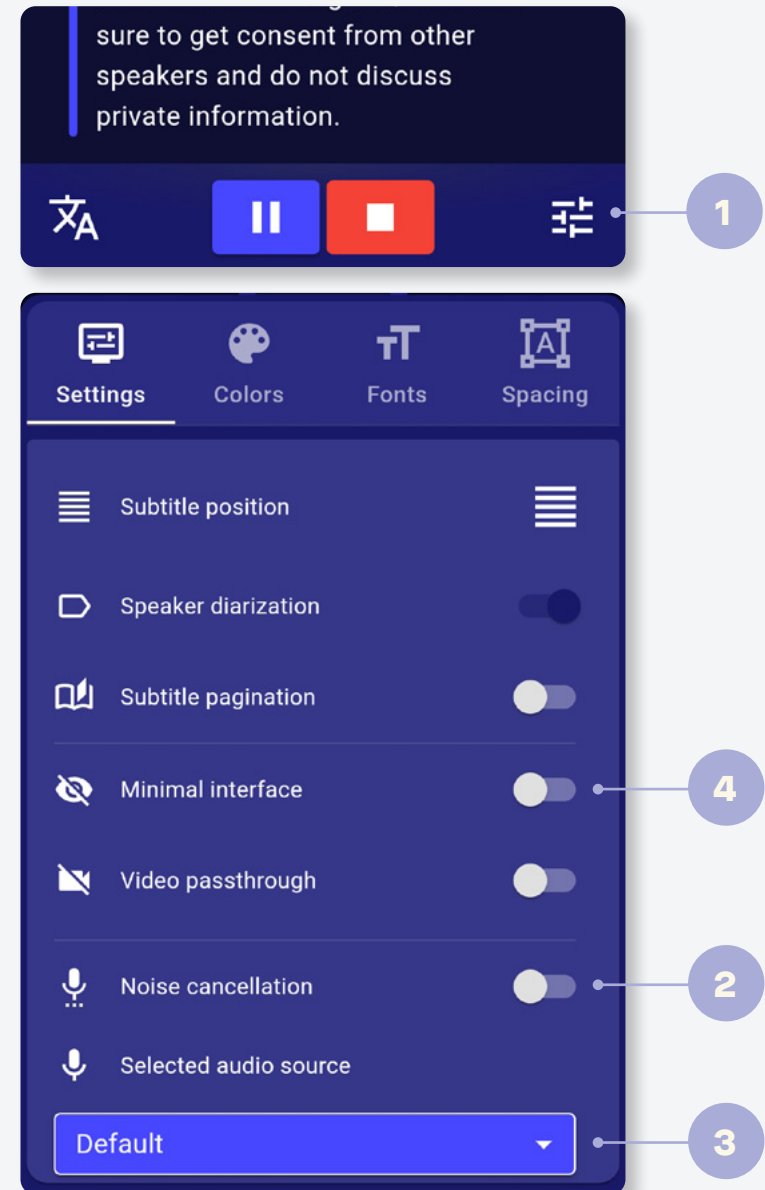

Languages

140+ languages and dialects


Picture yourself attending an event where the keynote speaker is speaking in a different language - or even a different dialect. With XRAI Glass®, you can instantly translate over 140+ languages and dialects, all in real-time.

Try it out

1. Select the language icon from the bottom bar.
2. From here, you can select the spoken language by clicking on the circle next to the language. You can also use the heart to add a language to your list of favorites.
3. Next, simply move to the **To** tab and select the language you'd like your subtitles to appear in. By default, your app will be set to No translation. After you're done, simply click off of the language menu to return to your conversation.



Read more...
How to translate

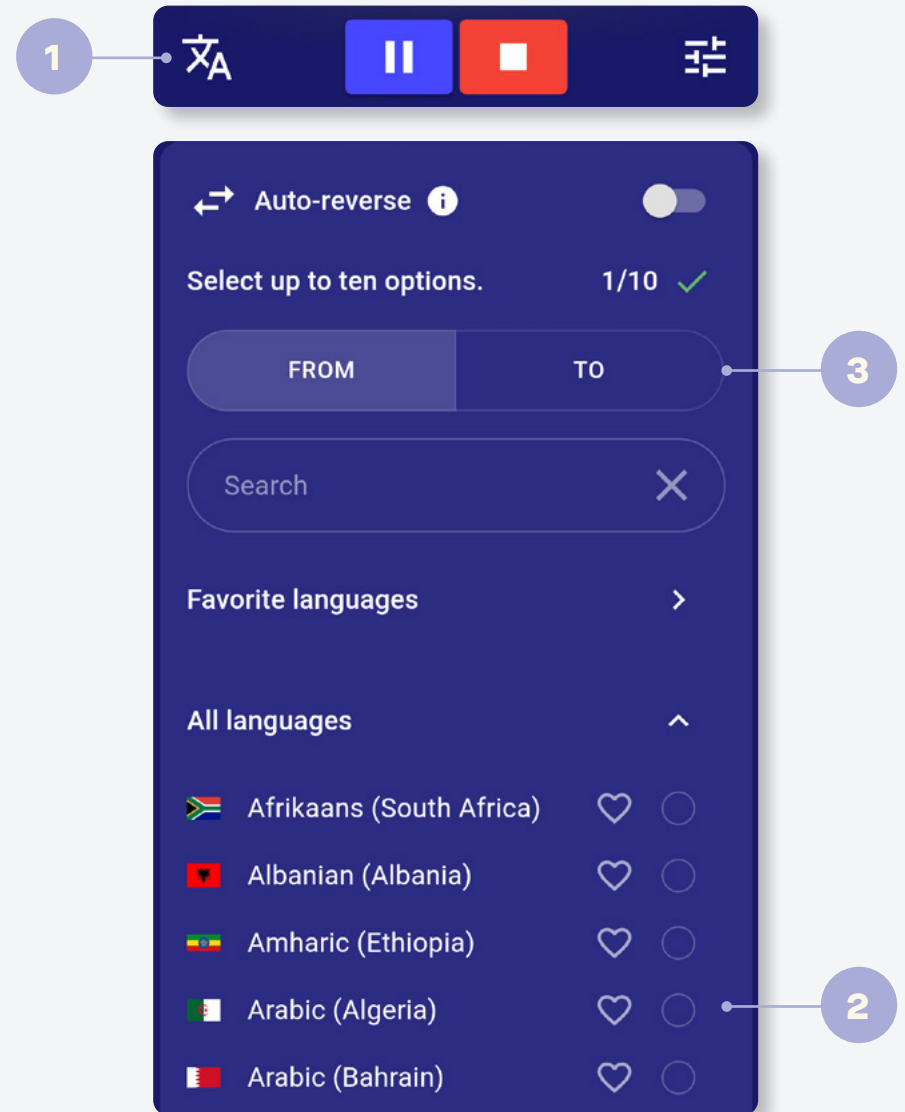

AI Assistant

Powered by ChatGPT


We've integrated ChatGPT into our app to deliver a deeper and more meaningful user experience. You can ask XRAI to change your speaker name, summarize your conversation, or even help you find the perfect recipe for dinner.

Try it out

1. Open up your settings by clicking on the user icon in the top right hand corner of the home screen.
2. Next, scroll down to find the **Conversational AI** option and toggle it on.
3. You'll then need to input your personal OpenAI GPT key to activate your AI assistant. To get your key, simply visit openai.com and register for a free account. Once registered, you'll be able to find your key in your user settings.



Read more...
How to set up the AI Assistant

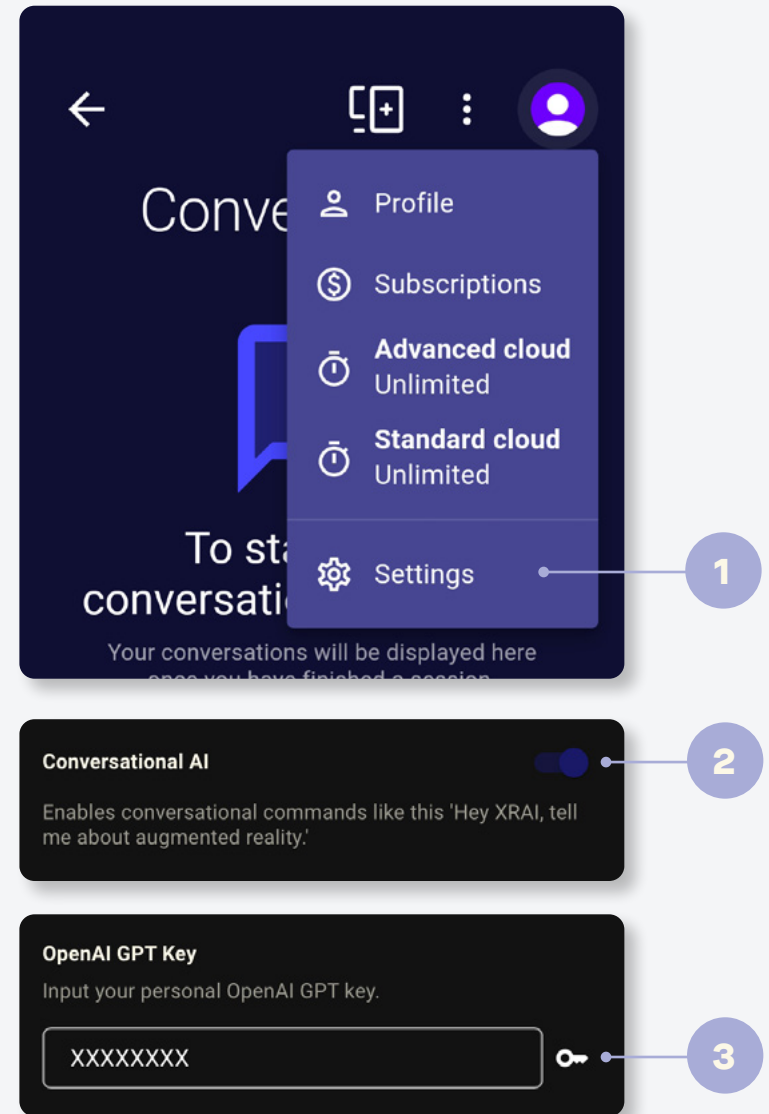

Meet themes


Your subtitles, your way

With XRAI Glass, it's all about you. Change the spacing for words, letters, margins, and more - on your phone or in a pair of smart glasses.

Try it out

1. Open up your quick settings menu and select **Colors**. From here, try selecting a new theme color.
2. Next, select **Fonts**. From here, try changing your font size (this works on both your mobile device and in a pair of smart glasses).
3. Finally, select **Spacing**. From here, you can change everything from your word, letter, and line spacing, to even your width and margin.

 [Read more...](#)
Quick settings menu

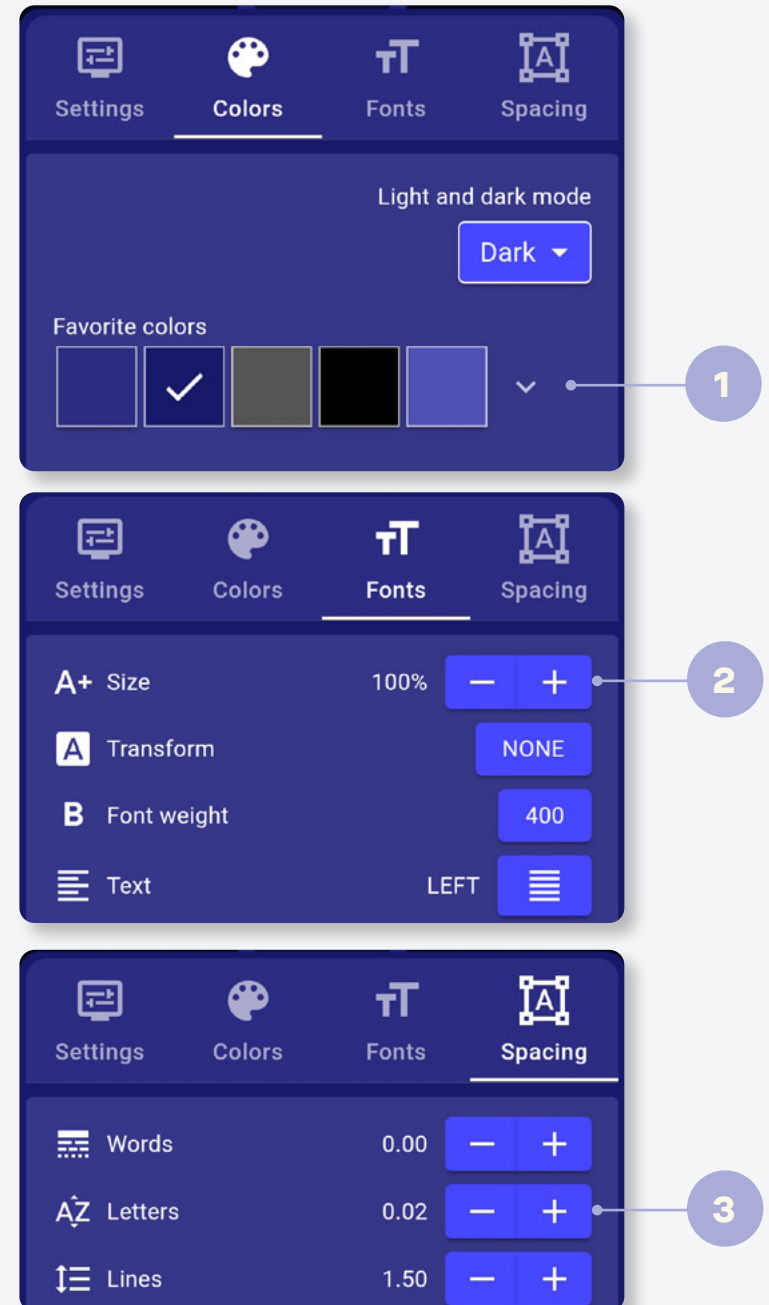

xrai
ar one

Life. Subtitled

Download the XRAI Glass app for free
and start subtitled your life today



Follow us on social for more tips and tricks to learn how to
get the most out of your XRAI Glass experience